

ultramat S

HIGH SPEED AMALGAMATOR

HOCHFREQUENZ MISCHGERÄT
AMALGAMADOR DE ALTA VELOCIDADE
AMALGAMADOR DE ALTA VELOCIDAD
VIBREUR ELECTRONIQUE A GRANDE VITESSE
VIBRATORE PER AMALGAMA AD ALTA VELOCITÀ
HIGH SPEED CAPSULESCHUDDER
HØJHASTIGHEDS-KAPSELRYSTER
HØYHASTIGHETS-KAPSELRYSTER
NOPEA KAPSELISEKOITIN
ΥΨΗΛΗΣ ΤΑΧΥΤΗΤΑΣ ΔΟΝΗΤΗΣ ΑΜΑΛΓΑΜΑΤΟΣ
WYSOKOOBROTOWY MIESZALNIK
NAGY SEBESSÉGŰ KAPSZULARÁZÓGÉP
SUURE KIIRUSEGA AMALGAAMISEGISTI
HITRI MEŠALEC AMALGAMA
AUGSTAS FREKVENCES AMALGAMATORS
DIDELIŲ APSUKŲ AMALGAMŲ MAIŠYTUVAS
VYSOKO RÝCHLOSTNÝ AMALGAMÁTOR
HITRI MEŠALEC AMALGAMA
高速アマルガムミキサー
高速銀汞調拌机



S stands for simple!

ultramats



High speed amalgamator


- Accurate and consistent trituration
- Low vibration and extremely quiet
- Simple to use - just 3 buttons
- Suitable for all your capsules
- Easy to clean
- Low power indicator and shut off function
- Dual voltage



ultramat S features:


So simple, just 3 simple buttons

There are three time setting options, 6, 8 and 10 seconds to mix all your encapsulated materials. The unit can be stopped at any time by pressing these buttons during operation or just by lifting the lid.

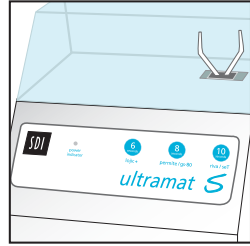

So easy to clean!

The hygienic one surface cover facilitates infection control cleaning and the non-porous plastic surface can be cleaned with any dental disinfectant.

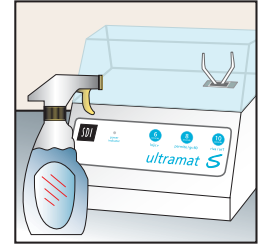


Accurate and consistent trituration

The Ultramat S features a microprocessor controlled oscillator timer for accurate and consistent trituration.




Dual voltage

The Ultramat S operates on voltages between 100-120V and 220-240V, 50 and 60Hz for compatibility with multiple power voltages and frequencies.

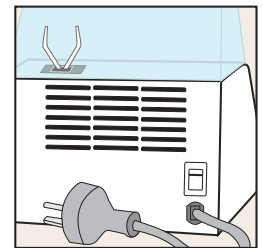


Low vibration and extremely quiet

Unlike some amalgamators, the Ultramat S has a low noise output. Minimal vibration of all external parts ensures it will not move from its standing position.

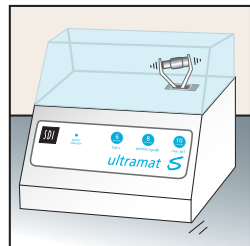


Low power indicator and shut off function

Should the power supply fall outside the required ranges of 100-120V and 220-240V, the power indicator will flash blue and the unit will not function until correct power is supplied to it. This prevents under-mixed materials.

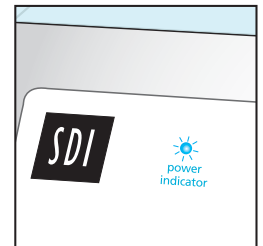


Suitable for all your capsules

The bendable forks can fit a multitude of round end capsules (amalgam capsules), and also those with protruding nozzles (glass ionomer cement capsules).

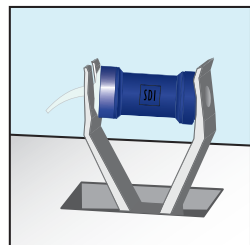



Safety feature

The Ultramat S will immediately stop mixing when the transparent lid is opened.

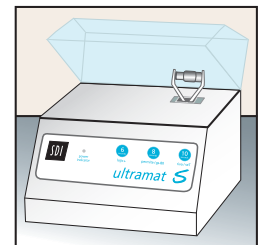

High energy output and low energy input

Possessing a mixing frequency of 4550 oscillations per minute (+/- 1%), the Ultramat S ensures complete mixing of any dental material, however it consumes relatively low power.

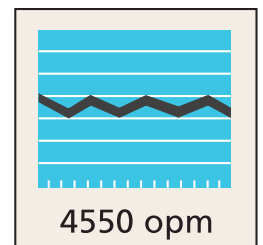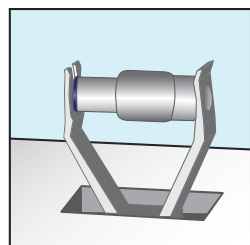

ultramats



Dimensions:

width	depth	height
217mm	173mm	180mm
8 1/2"	6 3/4"	7 1/8"

Voltage:

100 - 120 volts
220 - 240 volts
50 - 60 Hz

Weight:

2.43kg
5.4lbs

Ultramat S

(All countries - includes power cord
for region product is shipped to)
Reorder 5546058



Made in Australia by SDI Limited
Bayswater, Victoria 3153
Australia 1 800 337 003
Austria 00800 2255 734
Brazil 0800 770 1735
France 00800 022 55 734
Germany 0800 100 5759
Ireland 01 886 9577
Italy 800 780625
Japan 0120 98 2770
Spain 00800 022 55 734
United Kingdom 00800 022 55 734
USA & Canada 1 800 228 5166
www.sdi.com.au



M554620 B

+DO36M55462013

ated
ebsite